


To Apply:

1. Carefully start the expandable open end over the aid.
2. Gently slide and/or pull the Hearing Aid Sweat Band™ completely over aid up to and gathered around ear tube.
3. Cover the microphone area and let the Hearing Aid Sweat Band™ gather snugly around the ear tube.

Hand washing is recommended. Air dry.

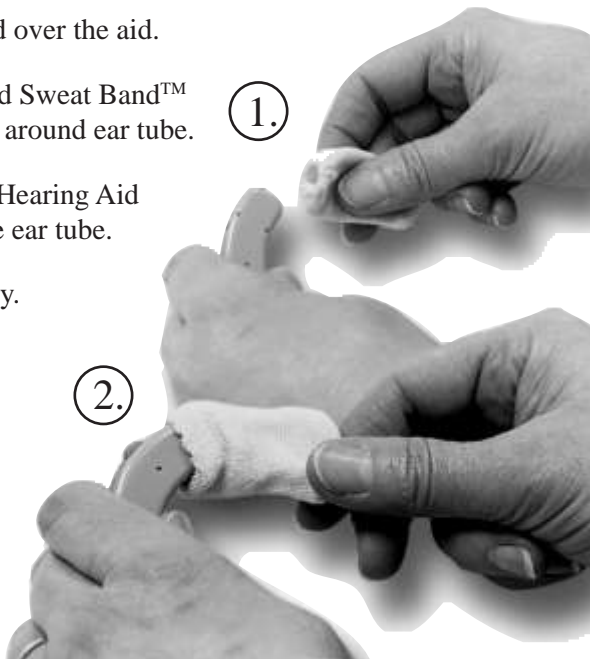

How To Apply

Hearing Aid SWEAT BAND™

Size Chart

Mini	1"
Small	1 1/4"
Medium	1 1/2"
Large	1 3/4"
Extra Large	2 1/8"
5XL (for FM systems)	3"
Slim Series	
Mini-Slim	1"
Small-Slim	1 1/4"
Medium-Slim	1 1/2"

Compliments of:

Bernafon Canada
1-800-265-8250

Hearing Aid SWEAT BAND™

ATTENTION: Disclaimers and Cautions

The intended use of the Hearing Aid Sweat Band™ is to repel moisture. It is not intended to protect the hearing aid from damage if submerged in water, in the shower, during shampooing of hair, or any other extreme or prolonged exposure to water, including rain and or excessive perspiration. When the Hearing Aid Sweat Band™ fails to repel moisture and begins to absorb moisture, it should be replaced with a dry one.

DO NOT USE WITH INFANTS
due to risk of swallowing or choking.

Copyright © 2007 VanB Enterprises
All rights reserved.